


Awarded by the Peter Abel Prize 2009

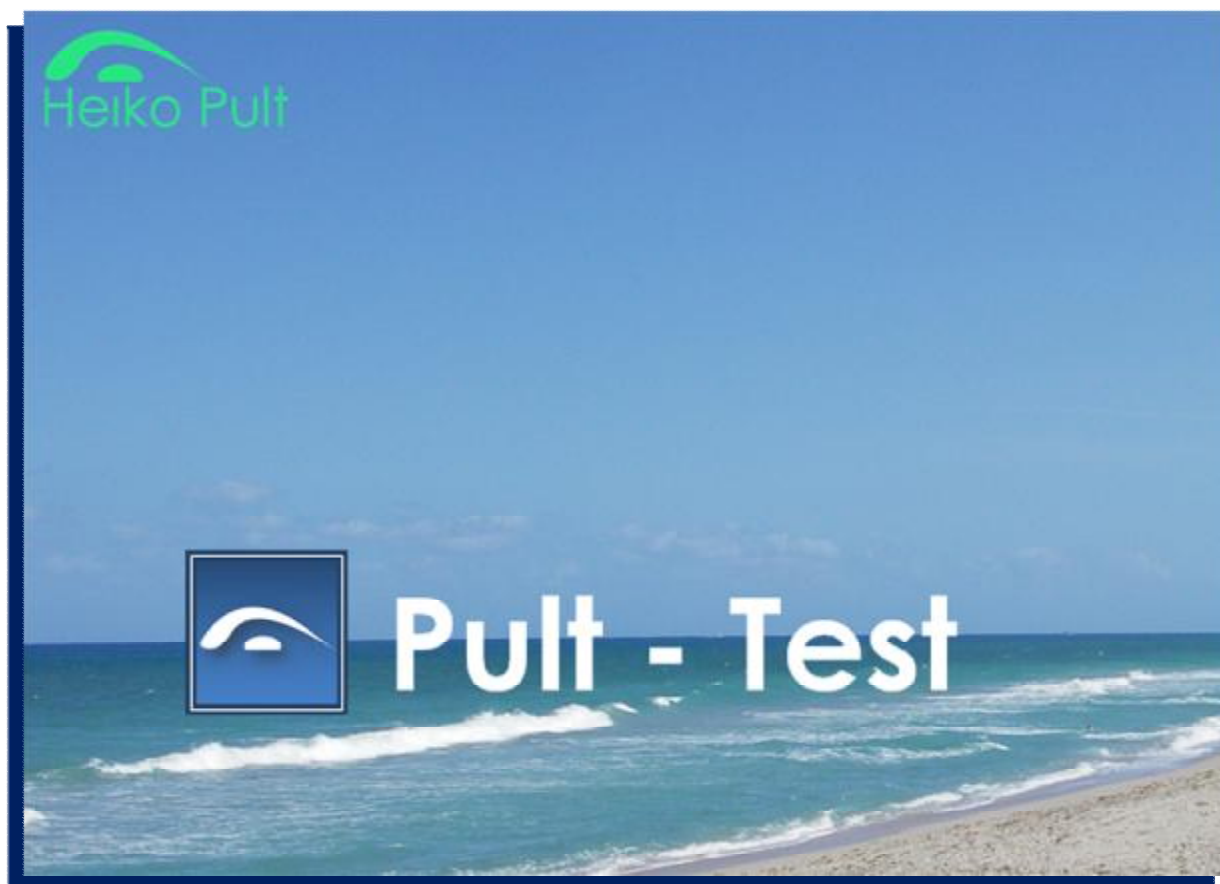

Manual

(12.2009 EN draft)

Content:

The Two Crucial Questions in Dry Eye:	3
Delivery Content:	4
Installation:.....	4
Requirements:.....	4
Language:.....	4
Modules:	5
Contact Lens Predicting Test I (P-Test I):.....	5
Contact Lens Predicting Test II (P-Test II):	5
Contact Lens Induced Dry Eye – Index (CLIDE-Index):	5
Which Test Should be Used?.....	5
The Modules?	6
Results:	7
P-Test I:	7
P-Test II:	7
CLIDE-Index:	7
Analyzes of the Results:.....	8
P-Test:.....	8
CLIDE-Index:	9
Working With The Program:.....	10
Screenshots:	10
Technique & Tests:	15
NIBUT:	15
LIPCOF:	15
Impressments:	21
Rights:	21
References:	22

The Two Crucial Questions in Dry Eye:

How can we predict contact lens wearing discomfort & dryness in new lens wearers?

How can experienced contact lens wearers be monitored and screened of dry eye symptoms?

Both might be the most important questions in clinical practice, since about 30-50% of all contact lens wearers claim dry eye symptoms.

However, we know from literature as well as from clinical practice, that dry eye symptoms are poorly predictable. In new contact lens wearers the clinician can react to discomfort after first some after cares, only; experienced contact lens wearers with increasing symptoms over years will mostly been overseen.

The new investigated method of Dr Heiko Pult and colleagues –named the PULT-Test- awarded by the Peter-Abel Prize 2009 of the German Contact Lens & Optometry Association (VDCO), will improve dry eye procedure in your contact lens practice dramatically.

This is a quick and exact method combining tear film tests and ocular signs with patient history resulting in an increased predictive ability (91%) of dry eye in lens wearers. This combination of tests is based on two special algorithms, easily calculated by the according *software*.

Modul I analyzes the risk of later dry eye in new contact lens wearer, Modul II diagnose dry eye in experienced lens wearers and is also able to monitor patients and to grade the degree of dryness and discomfort.

Enjoy it and increase your success in patient care!

Yours



Dr. Heiko Pult, PhD, MSc, FAAO, FBCLA, EAEO (founding member)

Delivery Content:

- CD
- USB-Dongle for activation of the program (copy protection).
- Manual as PDF

Installation:

Please copy the „exe“ file on your computers desktop and keep the dongle plugged in for lasting activation of the program at the computer.

The program will be started by double-clicking in that icon.



Requirements:

USB-Port

CD-R drive

Windows > `98, not Windows NT or MAC

Adobe Reader for the manual

Language:

This software is available in German and English. The current language settings of your computer will be recognized and all non-German set computer will use the English mode of the PULT-test.

Modules:

Contact Lens Predicting Test I (P-Test I):

This test evaluates the risk of later dry eye in new contact lens wearers (contact lens induced dry eye (CLIDE)). The P-Test measures the risk of later CLIDE. The practitioner can decide by that result about type of lens material and care solutions for improved fitting options in CLIDE. Additionally the patient can be educated about expected problems and procedure.

The P-Test I is a combination of the non-invasive analyzes of the tear film instability (NIBUT), evaluation of lid parallel conjunctival folds (LIPCOF) and history by use of the Ocular Surface Disease Index (OSDI)

Contact Lens Predicting Test II (P-Test II):

This test is similar to the PULT-Test I but without evaluation of the NIBUT, if the proper technical equipment is not available.

Contact Lens Induced Dry Eye – Index (CLIDE-Index):

The CLIDE-Index analyzes the dry status in experienced contact lens wearers. It diagnoses CLIDE and measures the level of dryness and discomfort. In any longitudinal changes in that index, the practitioner can react quickly. This test is a combination of contact lens related history and LIPCOF.

Which Test Should be Used?

It has to be distinguished between naive lens wearers and experienced lens wearers. The naive contact lens wearers can only be observed and diagnosed by the PULT-test and the experienced by the CLIDE-Index only.

Combining NIBUT + LIPCOF + OSDI, the P-Test I is slightly exacter than P-Test II.

However, this difference is minimal and P-test II is a perfect alternative if the NIBUT cannot be evaluated.

The CLIDE Index is for patient with at least 3 months experience in lens wear and should be included in every after care.

The Modules?

All modules are based on intense research. This resulted in proven algorithms of the optimal combination of subjective and objective dry eye tests.

P-Test I:

$$\text{CLIDE} = k_1 \times (\text{temporal LIPCOF} + \text{nasal LIPCOF}) + k_2 \times \text{NIBUT} + k_3 \times \text{OSDI} - K$$

Increased LIPCOF and/or increased OSDI scores are indicators of later CLIDE. However, longer NIBUT values decrease that degree of risk.

P-Test II:

$$\text{CLIDE} = k_1 \times (\text{temporal LIPCOF} + \text{nasal LIPCOF}) + k_2 \times \text{OSDI} - K$$

Increased LIPCOF and/or increased OSDI scores are indicators of later CLIDE.

CLIDE-Index:

$$\text{CLIDE} = k_1 \times \text{dryness} - k_2 \times \text{grittiness and scratchiness} + k_3 \times (\text{temporal LIPCOF} + \text{nasal LIPCOF})$$

Increased LIPCOF and/or increased symptoms of dryness are indicator of CLIDE. However increases sensation of grittiness and scratchiness are not dry eye indicators! The latter are more related to too loose fitted contact lenses, deposits on the contact lens or lens defects, etc..

Therefore, in the CLIDE-Index module discomfort is asked too, even though it is not part of the formula. This is an additional indicator of the overall success of the contact lens fitting.

Results:

P-Test I:

OSDI: History evaluated by the Ocular Surface Disease Index.

NIBUT: Median of three measurements of the tear film instability.

Temporal LIPCOF: Degree of LIPCOF (0-3) of the temporal area of observation.

Nasal LIPCOF: Degree of LIPCOF (0-3) of the nasal area of observation.

Probability of CLIDE: That value shows the probability of later dry eye symptoms in lens wear. E.g. shows how likely the patient will mention / claim dry eye symptoms in later lens wear.

PPV - CLIDE: This is the positive predictive value of CLIDE by a prevalence of 43%.

Screening: "CLIDE –" has to be interpreted that the patient will not be in the group of later symptomatic lens wearers, "CLIDE +" indicates the potential symptomatics.

P-Test II:

OSDI: History evaluated by the Ocular Surface Disease Index.

Temporal LIPCOF: Degree of LIPCOF (0-3) of the temporal area of observation.

Nasal LIPCOF: Degree of LIPCOF (0-3) of the nasal area of observation.

Probability of CLIDE: That value shows the probability of later dry eye symptoms in lens wear. E.g. shows how likely the patient will mention / claim dry eye symptoms in later lens wear.

PPV - CLIDE: This is the positive predictive value of CLIDE by a prevalence of 43%.

Screening: "CLIDE –" has to be interpreted that the patient will not be in the group of later symptomatic lens wearers, "CLIDE +" indicates the potential symptomatics.

CLIDE-Index:

CLIDE-Index: Degree of CLIDE in experienced lens wearers

Screening: "CLIDE –" has to be interpreted that the patient will not be in the group of current symptomatic lens wearers, "CLIDE +" indicates the symptomatics.

Analyzes of the Results:

P-Test:

Please concentrate on “probability” in that test. The PPV get closer to the “probability” in higher degrees, in lower probability values, the PPV is higher. That’s because the PPV is calculated by the 43% prevalence of CLIDE. The PPV is more useful in research projects than clinical practice.

A probability of CIDE of $\geq 70\%$ should result into a proper CLIDE fitting. In lower values (50-60%) later symptoms are not very likely, however please decide individually.

Nevertheless, it has to be noted, that new contact lens wearers are very often highly motivated. Therefore symptoms, which will be experienced from those patients in the first months, will be accepted. Practice tells us, that if the positive motivating factors are decreasing, the patient starts to claim these symptoms of discomfort and dryness in later stages. Therefore try to keep it save. Fit proper lenses in CLIDE risk candidates and observe seriously in following after cares using the CLIDE-Index.




CLIDE-Index:

The CLIDE-Index measures the degree of CLIDE in a scale from 0 to 31. Contact lens wearers with a degree of ≥ 18.04 are very likely CLIDE patients. Even they does not know, because high motivation and acceptance of some symptoms. Many contact lens wearers believe that lenses have to hurt in some way. Therefore keep those patients in short follow up intervals. General, any change of $\geq \pm 5.5$ is abnormal.

Important: There are many options to improve contact lens wearing comfort. Therefore, a contact lens wearer can present with increased LIPCOF but no symptoms in the CLIDE-Index, since he has got the proper “treatment”.


Example: A new lens wearer appears as risk candidate and is fitted with best type of lens and care regimes. Then he will still have increased LIPCOF but no symptoms in lens wear. The CLIDE-Index result shows higher LIPCOF but low symptoms and likely CLIDE +. In those cases you have to look very seriously on to the symptoms results in the CLIDE-Index. However, do not forget that this is still a risk candidate who has to be observed in short intervals. Therefore, always compare the CLIDE-Index results with the patients treatments and keep the risk in your mind.

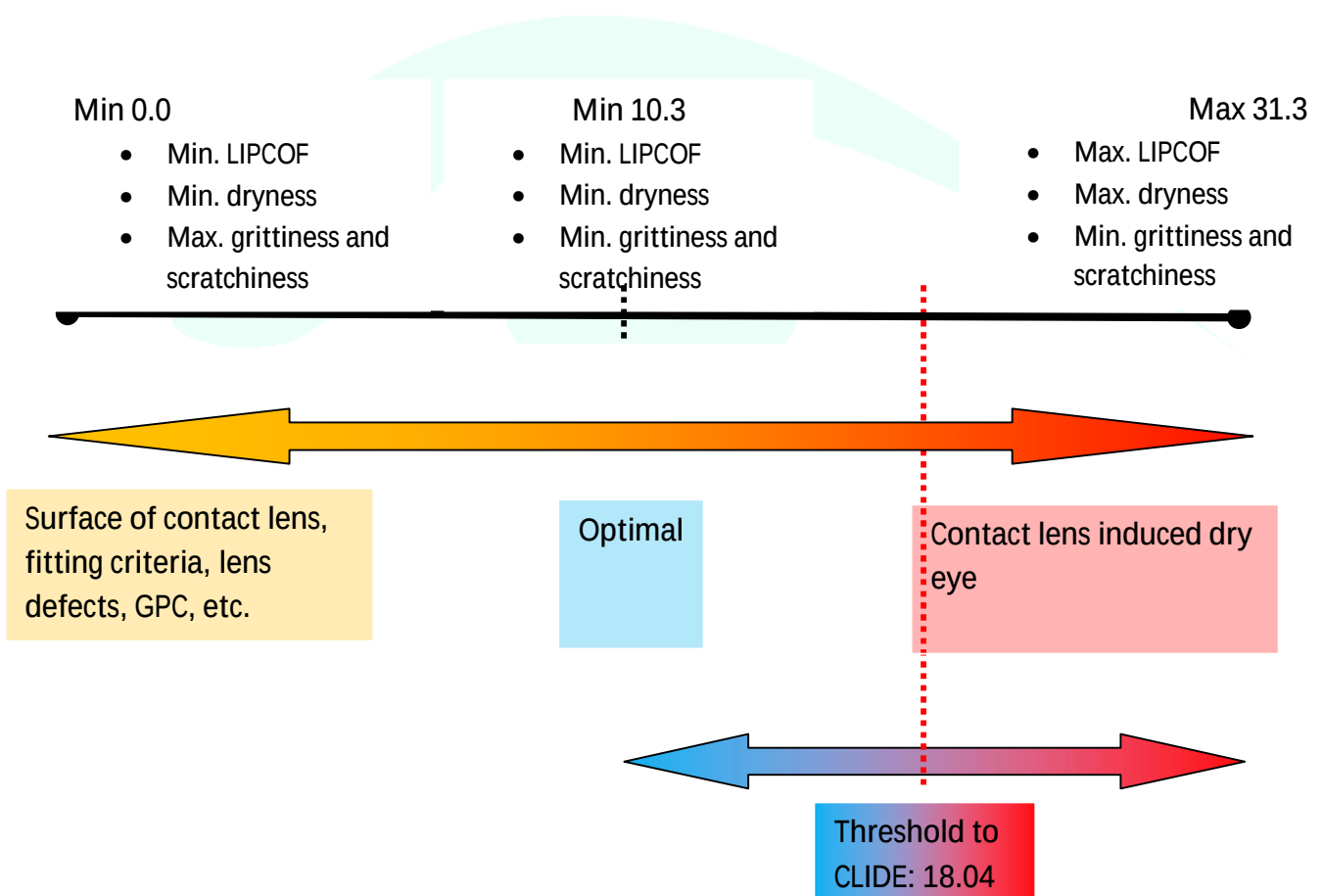



Working With The Program:

All data will be saved to name and date of birth. At the initial saving process you have to create a folder where the data have to be saved to, best option is PULT-Test Data. Every next patient will be saved to that one automatically.

Screenshots:

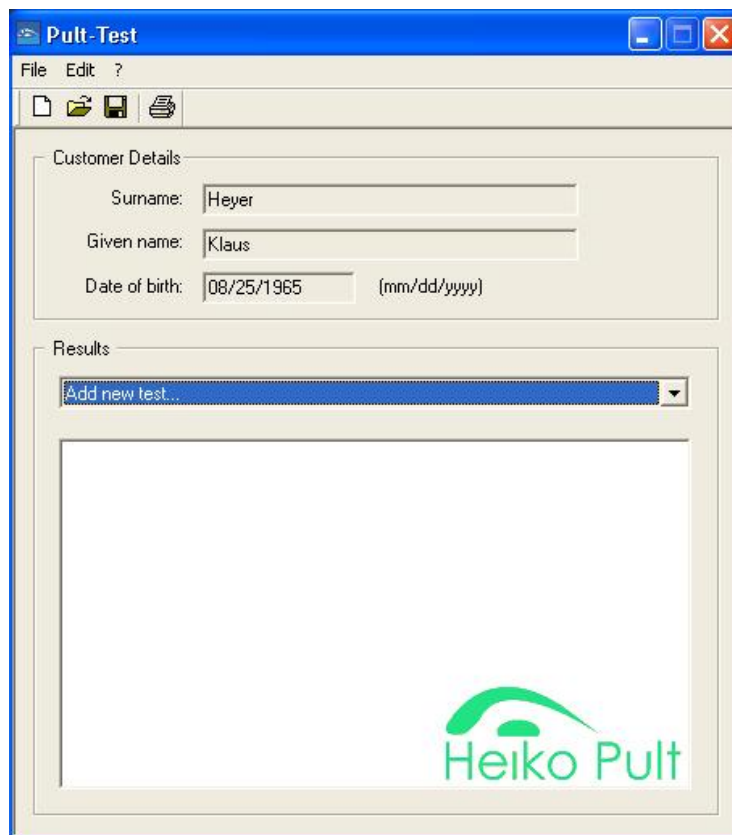

Open existing patient data or fill in name and date of birth in new patients.

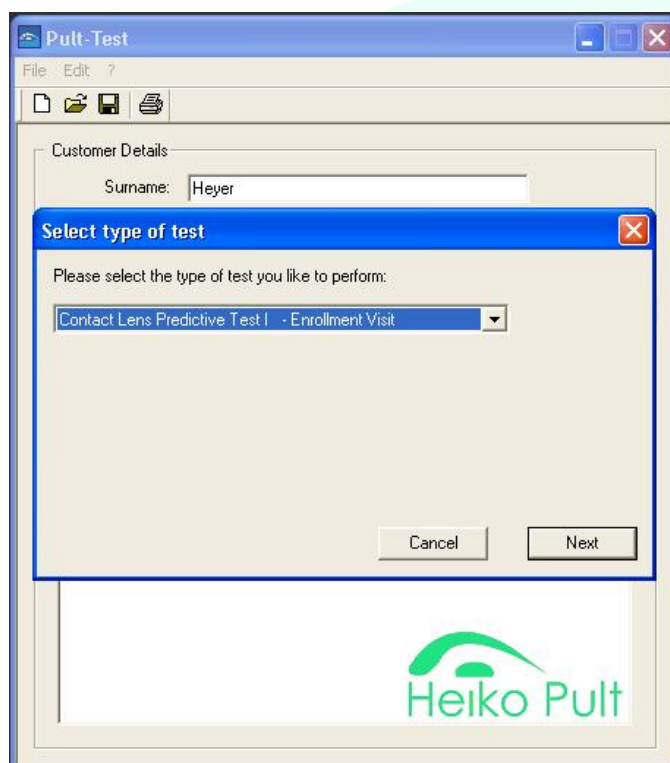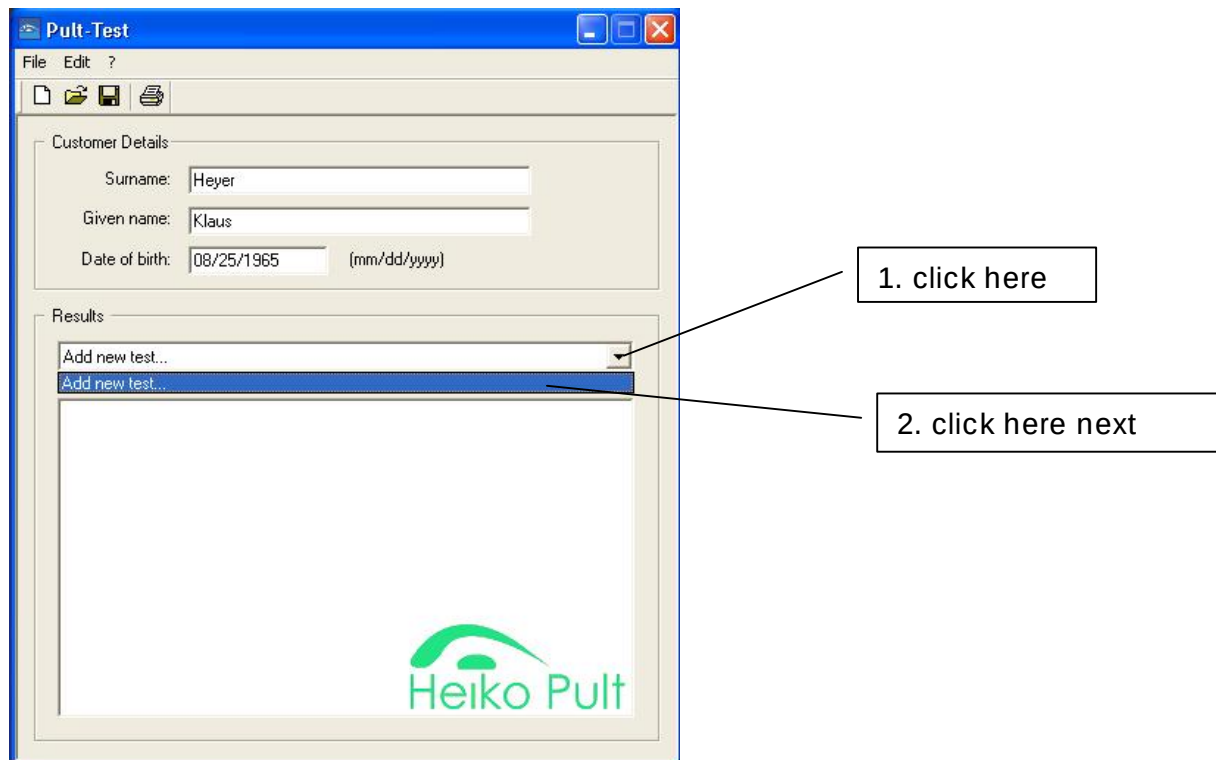

Choose the proper module.

Pult-Test

File Edit ?

Customer Details

Surname:

Objective Evaluation:

NIBUT: 1. 2. 3. (sec.)

Temporal LIPCOF:

Nasal LIPCOF:

0
1
2
3

Back Next



Fill in data and click "next".

Pult-Test

File Edit ?

Customer Details

Surname:

Subjective Evaluation 1

Have you experienced any of the following during the last week?

Eyes that are sensitive to light?

Eyes that feel gritty?

Painful or sore eyes?

Blurred vision?

Poor vision?

Back Next



Fill in data and click "next".

Pult-Test

File Edit ?

Customer Details

Surname: Heyer

Results

The results for the Contact Lens Predictive Test I - Enrollment Visit:

OSDI Score:	16.7
Probability of later CLIDE:	100%
PPV-CLIDE:	100%
Screening:	CLIDE +

Back Finish

Heiko Pult

Results are shown, then end by clicking on “finish”.

Pult-Test

File Edit ?

Customer Details

Surname: Heyer

Given name: Klaus

Date of birth: 08/25/1965 (mm/dd/yyyy)

Results

11/01/2010 Contact Lens Predictive Test I - Enrollment Visit

OSDI Score:	16.7
NIBUT (Median):	9.2
LIPCOF temporal:	1
LIPCOF nasal:	1
Probability of CLIDE:	100%
PPV-CLIDE:	100%
Screening:	CLIDE +

Heiko Pult

All results can be seen then; you also can print them or open prior results to compare.

Pult-Test

File Edit ?

Existing files can be opened here

Customer Details

Surname:

Given name:

Date of birth: (mm/dd/yyyy)

Results

Add new test...

17/02/2010	Contact Lens Dry Eye Index	- Verlaufskontrolle
23/10/2009	Contact Lens Dry Eye Index	- Verlaufskontrolle
23/04/2009	Contact Lens Predictive Test I	- Anpassung

Discomfort: 2.0

Dryness: 4.0

Grittiness: 0.0

CLIDE-Index: 18.1

Screening: CLIDE +

Heiko Pult

In an addiotinal after care of that patient you can proceed with the next test.
All data are archived.

Technique & Tests:

Not to be biased, please always start with objective evaluation.

The quality of the results depends on the exactness of observation. Therefore all tests are described as following.


NIBUT:

NIBUT is the non-invasive break up time, to be evaluated by use of the Tearscope™ of Keeler or alternatively using a topographer. NIBUT should be the first proceeded observation at the patients eye, since any touching or glaring impacts the tear film.

LIPCOF:

LIPCOF are lid parallel conjunctival folds. It is very important to observe the right folds at the correct area.

- Without worn contact lenses (can be observed immediately after removing lenses)
- Patient has to look straight forward
- No use of vital dyes like fluorescein
- 18-24x magnification of the slit-lamp microscope
- Go perpendicular from the lateral limbus down to the lower lid to view at the correct area of observation.

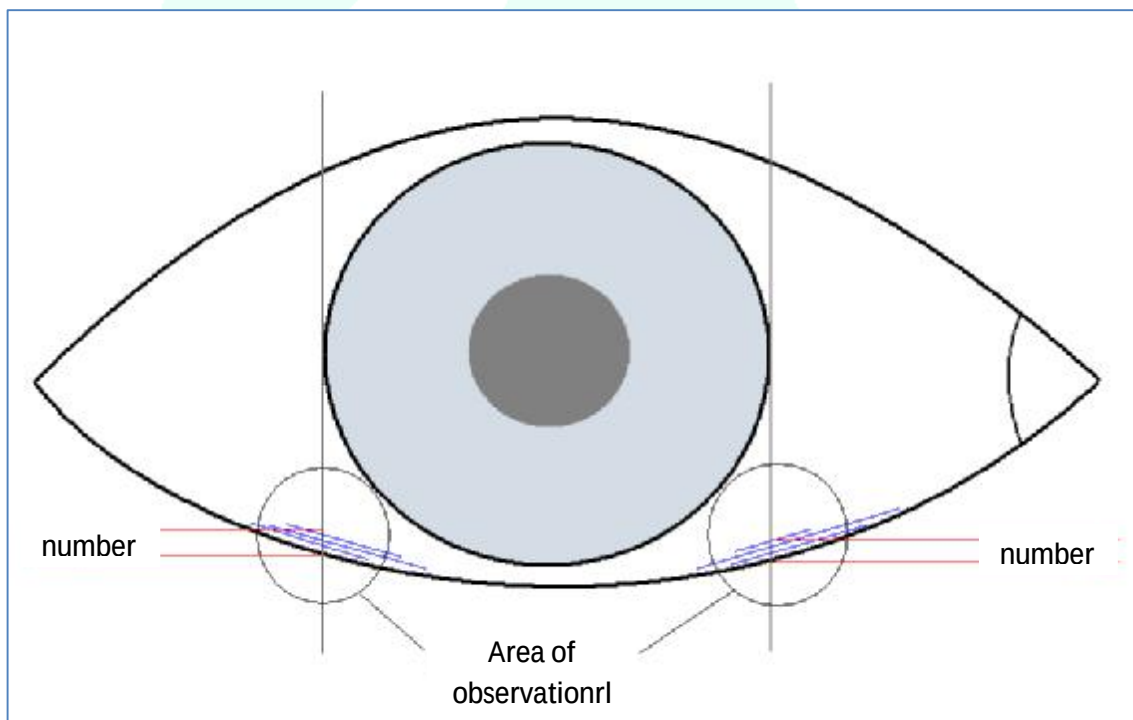


- Optimized grading scale:


	LIPCOF Grad
No permanent, parallel conjunctival fold	0
One permanent, parallel conjunctival folds	1
Two permanent, parallel conjunctival folds	2
More than two permanent, parallel conjunctival folds	3

Please differentiate between lid parallel conjunctival folds, and disrupted micro-folds or conjunctival flaps!




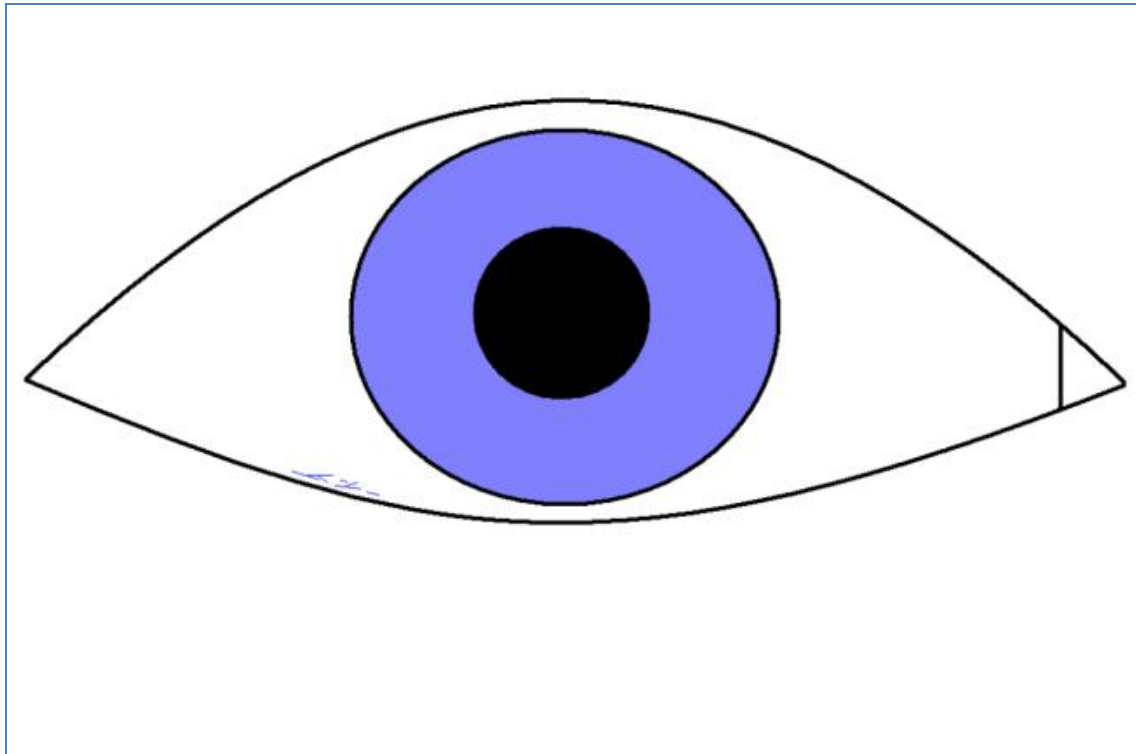


Some disrupted micro-folds, no permanent parallel conjunctival fold:
LIPCOF degree 0

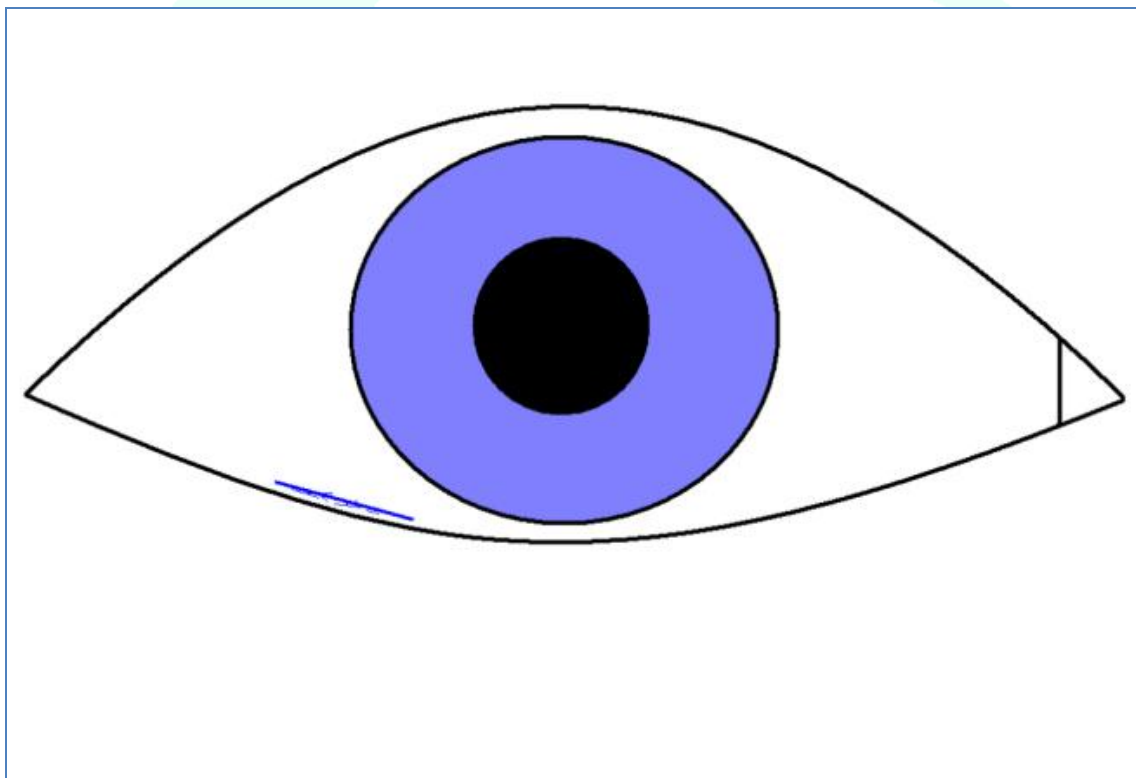


One clear, permanent and parallel conjunctival fold plus some disrupted micro-folds:
LIPCOF degree 1

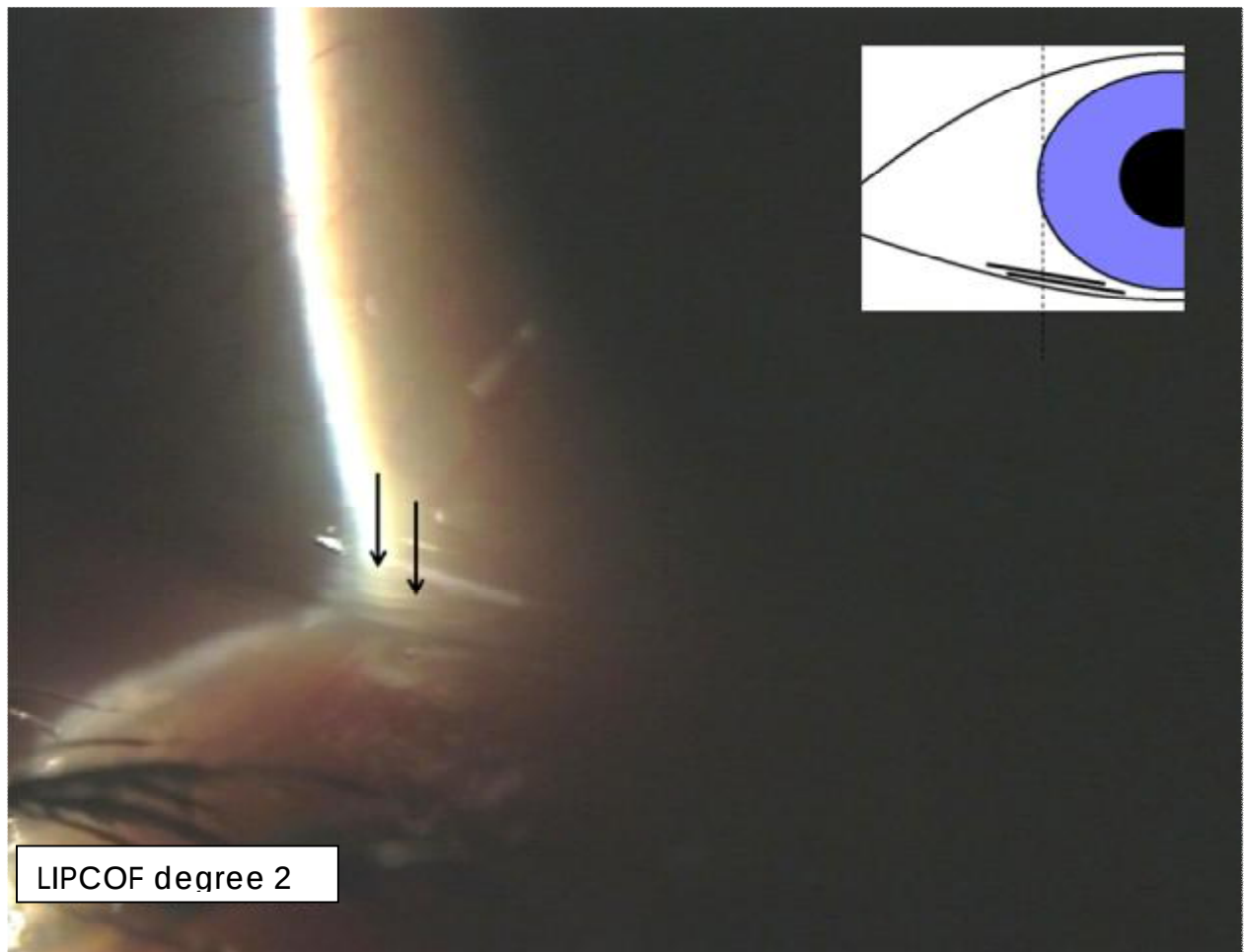

Two clear, permanent and parallel conjunctival folds: LIPCOF degree 2

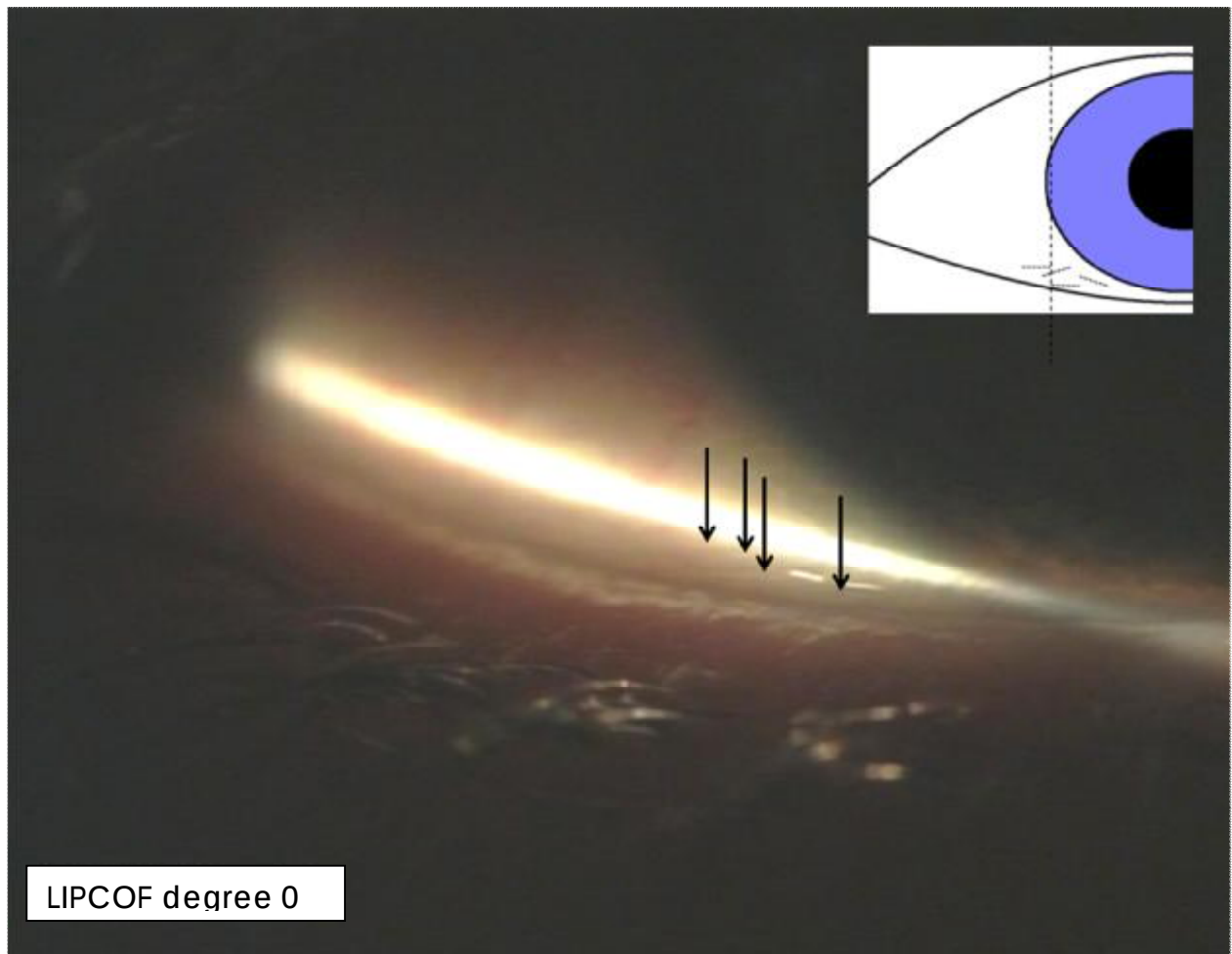

Four disrupted micro-folds but no clear permanent conjunctival fold:
LIPCOF degree 0

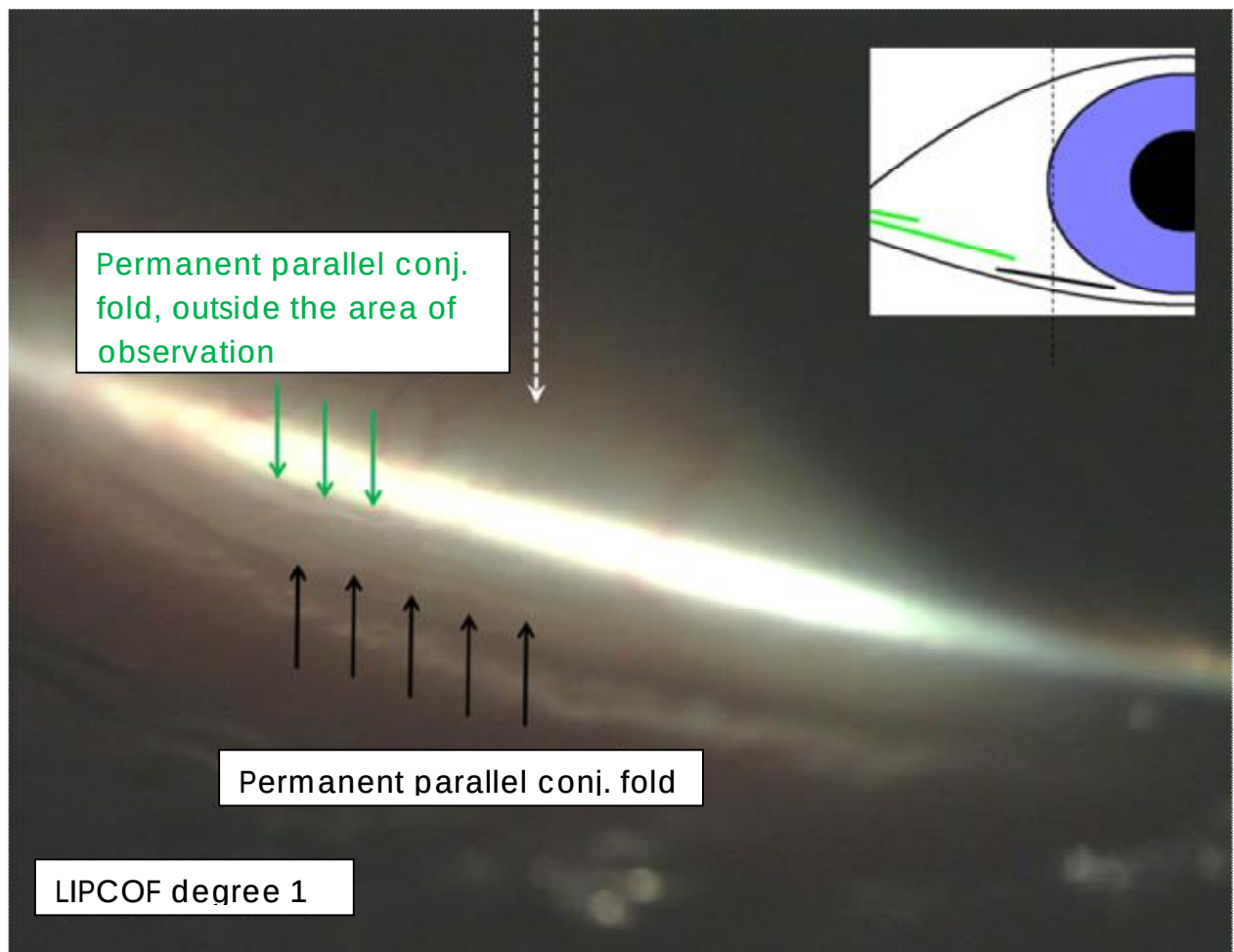

One clear, permanent conjunctival fold (black arrows) plus a second one outside the area of observation (green arrows): LIPCOF degree 1

Caution: As more you tend to the lid corners as higher the LIPCOF degrees appears. However, this would lead to over-classification. Please stay at the correct area of observation (black arrows).

Impressments:

Dr. Heiko Pult
Steingasse 15
69469 Weinheim
Germany
+49 (0) 174 90 250 90
ovr@heiko-pult.de
www.heiko-pult.de

Rights:

Copyright (C) 2009 - Dr. Heiko Pult
All rights reserved, patent pending

References:

- Begley, C. G., Caffery, B., Nichols, K. K. and Chalmers, R. (2000). Responses of contact lens wearers to a dry eye survey. *Optom Vis Sci* 77, 40-46.
- Begley, C. G., Chalmers, R. L., Mitchell, G. L., *et al.* (2001). Characterization of ocular surface symptoms from optometric practices in North America. *Cornea* 20, 610-618.
- Berry, M., Pult, H., Purslow, C. and Murphy, P. J. (2008). Mucins and ocular signs in symptomatic and asymptomatic contact lens wear. *Optom Vis Sci* 85, E930-938.
- Bron, A. J. (2001). Diagnosis of dry eye. *Surv Ophthalmol* 45 Suppl 2, S221-226.
- Chalmers, R. L. and Begley, C. G. (2006). Dryness symptoms among an unselected clinical population with and without contact lens wear. *Cont Lens Anterior Eye* 29, 25-30.
- Dillehay, S. M. (2007). Does the level of available oxygen impact comfort in contact lens wear?: A review of the literature. *Eye Contact Lens: Science & Clinical Practice* 33, 148-155.
- Döhr, C. (2005). Contact lens wearing comfort. In: *Contact '05, VDC Congress*, Augsburg, Germany.
- Dumbleton, K. A., Chalmers, R. L., Richter, D. B. and Fonn, D. (2001). Vascular response to extended wear of hydrogel lenses with high and low oxygen permeability. *Optom Vis Sci* 78, 147-151.
- Fonn, D. (2007). Targeting contact lens induced dryness and discomfort: What properties will make lenses more comfortable. *Optom Vis Sci* 84, 279-285.
- Fonn, D. and Dumbleton, K. (2003). Dryness and discomfort with silicone hydrogel contact lenses. *Eye Contact Lens* 29, S101-104; discussion S115-108, S192-104.
- Garson, G. D. (Accessed February 12, 2008). PA765 Multivariate Analysis in Public Administration Available at: <http://www2.chass.ncsu.edu/garson/pa765/statnote.htm>.
- Glasson, M. J., Stapleton, F., Keay, L., Sweeney, D. and Willcox, M. D. P. (2003). Differences in clinical parameters and tear film of tolerant and intolerant contact lens wearers. *Invest Ophthalmol Vis Sci* 44, 5116-5124.
- Guillon, J. P. (1998). Use of the Tearscope Plus and attachments in the routine examination of the marginal dry eye contact lens patient. *Adv Exp Med Biol* 438, 859-867.
- Guillon, M. P. F. and Maissa, C. P. (2005). Dry eye symptomatology of soft contact lens wearers and nonwearers. *Optom Vis Sci* 82(9), 829-834.
- Höh, H., Schirra, F., Kienecker, C. and Ruprecht, K. W. (1995). Lid-parallel conjunctival folds are a sure diagnostic sign of dry eye. *Ophthalmologe* 92, 802-808.
- lii, G. W., Anderson, R. T. and Osborn, K. E. (2007). The effect of senofilcon A contact lenses compared to habitual contact lenses on ocular discomfort during exposure to a controlled adverse environment. *Curr Med Res Opin*.
- Lemeshow, S. and Hosmer, D. W. (1982). A Review of Goodness of Fit Statistics for Use in the Development of Logistic-Regression Models. *American Journal of Epidemiology* 115, 92-106.
- Markoulli, M., Carnt, N., Jalbert, I., Keay, L., Naduvilath, T. and Papas, E. (2007). Resolution and Clinical Characteristics of Conjunctival "Flaps". *Invest Ophthalmol Vis Sci* 48, ARVO E-Abstract: 5391.
- Mcmonnies, C. W., Chapman-Davies, A. and Holden, B. A. (1982). The vascular response to contact lens wear. *Am J Optom Physiol Opt* 59, 795-799.
- Michel, M., Sickenberger, W. and Pult, H. (2009). The effectiveness of questionnaires in the determination of Contact Lens Induced Dry Eye. *Ophthalmic Physiol Opt* 29, 479-486.
- Morgan, P. B. (2001). Is the UK contact lens market healthy? *Optician* 221:5795, 22-26.
- Morgan, P. B. and Efron, N. (2006). A decade of contact lens prescribing trends in the United Kingdom (1996-2005). *Cont Lens Anterior Eye* 29, 59-68.
- Morgan, P. B. and Efron, N. (2008). Demographics of UK contact lens prescribing. *Cont Lens Anterior Eye* 31, 50-51.
- Nichols, J. J., Mitchell, G. L., Nichols, K. K., Chalmers, R. and Begley, C. (2002). The performance of the contact lens dry eye questionnaire as a screening survey for contact lens-related dry eye. *Cornea* 21, 469-475.

- Nichols, J. J. and Sinnott, L. T. (2006). Tear film, contact lens, and patient-related factors associated with contact lens-related dry eye. *Invest Ophthalmol Vis Sci* 47, 1319-1328.
- Nichols, K. K., Mitchell, G. L. and Zadnik, K. (2004). The repeatability of clinical measurements of dry eye. *Cornea* 23, 272-285.
- Nichols, K. K. and Zadnik, K. (2002). The repeatability of diagnostic tests and surveys in dry eye. *Adv Exp Med Biol* 506, 1171-1175.
- Osborn, K. and Veys, J. (2005). A new silicone hydrogel lens for contact lens-related dryness. Part 1. *Optician* 229.
- Ousler, G. W., 3rd, Anderson, R. T. and Osborn, K. E. (2008). The effect of senofilcon A contact lenses compared to habitual contact lenses on ocular discomfort during exposure to a controlled adverse environment. *Curr Med Res Opin* 24, 335-341.
- Papas, E. (1998). On the relationship between soft contact lens oxygen transmissibility and induced limbal hyperaemia. *Exp Eye Res* 67, 125-131.
- Papas, E. B., Vajdic, C. M., Austen, R. and Holden, B. A. (1997). High-oxygen-transmissibility soft contact lenses do not induce limbal hyperaemia. *Curr Eye Res* 16, 942-948.
- Pflugfelder, S. C., Tseng, S. C., Sanabria, O., et al. (1998). Evaluation of subjective assessments and objective diagnostic tests for diagnosing tear-film disorders known to cause ocular irritation. *Cornea* 17, 38-56.
- Pritchard, N. (2001). How can we avoid CL drop-outs? *Optician* 5825:222, 14-18.
- Pritchard, N., Fonn, D. and Brazeau, D. (1999). Discontinuation of contact lens wear: a survey. *Int Contact Lens Clin* 26, 157-162.
- Pult, H. (2008). The predictive ability of clinical tests for dry eye in contact lens wear. In: *School of Optometry and Vision Sciences*, Cardiff University, Cardiff.
- Pult, H., Murphy, P. J. and Purslow, C. (2009a). A Novel Method to Predict Dry Eye Symptoms in New Contact Lens Wearers. *Optom Vis Sci* 86, E1042-1050.
- Pult, H., Murphy, P. J., Purslow, C., Nyman, J. and Woods, R. L. (2008a). Limbal and bulbar hyperaemia in normal eyes. *Ophthalmic Physiol Opt* 28, 13-20.
- Pult, H., Purslow, C., Berry, M. and Murphy, P. J. (2008b). Clinical tests for successful contact lens wear: relationship and predictive potential. *Optom Vis Sci* 85, E924-929.
- Pult, H., Purslow, C. and Murphy, P. J. (2007). An update on the objective and subjective evaluation of contact lens related dry eye. *Optom Today* 47, 42-46.
- Pult, H., Purslow, C. and Murphy, P. J. (2009b). CLIDE-Index: A novel method to diagnose and measure Contact Lens Induced Dry Eye (CLIDE)? In: *107. DOG Kongress*, Leipzig, Germany.
- Richdale, K., Sinnott, L., Skadahl, E. and Nichols, J. J. (2007). Frequency of and factors associated with contact lens dissatisfaction and discontinuation. *Cornea* 26(2), 168-174.
- Ruston, D. and Meyler, J. (2005). A new silicone hydrogel lens. Part 2. *Optician* 230.
- Santodomingo-Rubido, J., Wolffsohn, J. and Gilmartin, B. (2008). Conjunctival epithelial flaps with 18 months of silicone hydrogel contact lens wear. *Eye Contact Lens* 34, 35-38.
- Schafer, J. (2006). Choosing lens materials to solve dryness complaints. *Contact Lens Spectrum* 21; 3, 23.
- Schafer, J., Mitchell, G. L., Chalmers, R. L., et al. (2007). The stability of dryness symptoms after refitting with silicone hydrogel contact lenses over 3 years. *Eye Contact Lens* 33, 247-252.
- Schiffman, R. M., Christianson, M. D., Jacobsen, G., Hirsch, J. D. and Reis, B. L. (2000). Reliability and validity of the Ocular Surface Disease Index. *Arch Ophthalmol* 118, 615-621.
- Schirra, F., Hoh, H., Kienecker, C. and Ruprecht, K. W. (1998). Using LIPCOF (lid-parallel conjunctival fold) for assessing the degree of dry eye, it is essential to observe the exact position of that specific fold. *Adv Exp Med Biol* 438, 853-858.
- Sickenberger, W., Pult, H. and Sickenberger, B. (2000). LIPCOF and contact lens wearers - A new tool of forecast subjective dryness and degree of comfort of contact lens wearers. *Contactologia* 22, 74-79.
- Sindt, C. W. and Longmuir, R. A. (2007). Contact lens strategies for the patient with dry eye. *Ocul Surf* 5, 294-307.

- Thota, S., Perrigin, J., Miller, W., Leach, N., Bergmanson, J. and Back, A. (2006). Conjunctival Flaps in Silicone Hydrogel Lens Wearers. *Invest Ophthalmol Vis Sci* 47, ARVO E-Abstract: 82.
- Van Belle, G., Fischer, L. D., Heagerty, P. J. and Lumley, T. (2004). *Biostatistics: A methodology for the health sciences*. John Wiley & Sons, Inc., Hoboken, New Jersey, USA
- Weed, K., Fonn, D. and Potvin, R. (1993). Discontinuation of contact lens wear. *Optom Vis Sci* 70 (Suppl):140.
- Young, G., Riley, C. M., Chalmers, R. L. and Hunt, C. (2007). Hydrogel lens comfort in challenging environments and the effect of refitting with silicone hydrogel lenses. *Optom Vis Sci* 84, 302-308.

